
User manual for the knee pain simulator

User manual for the knee pain simulator

Components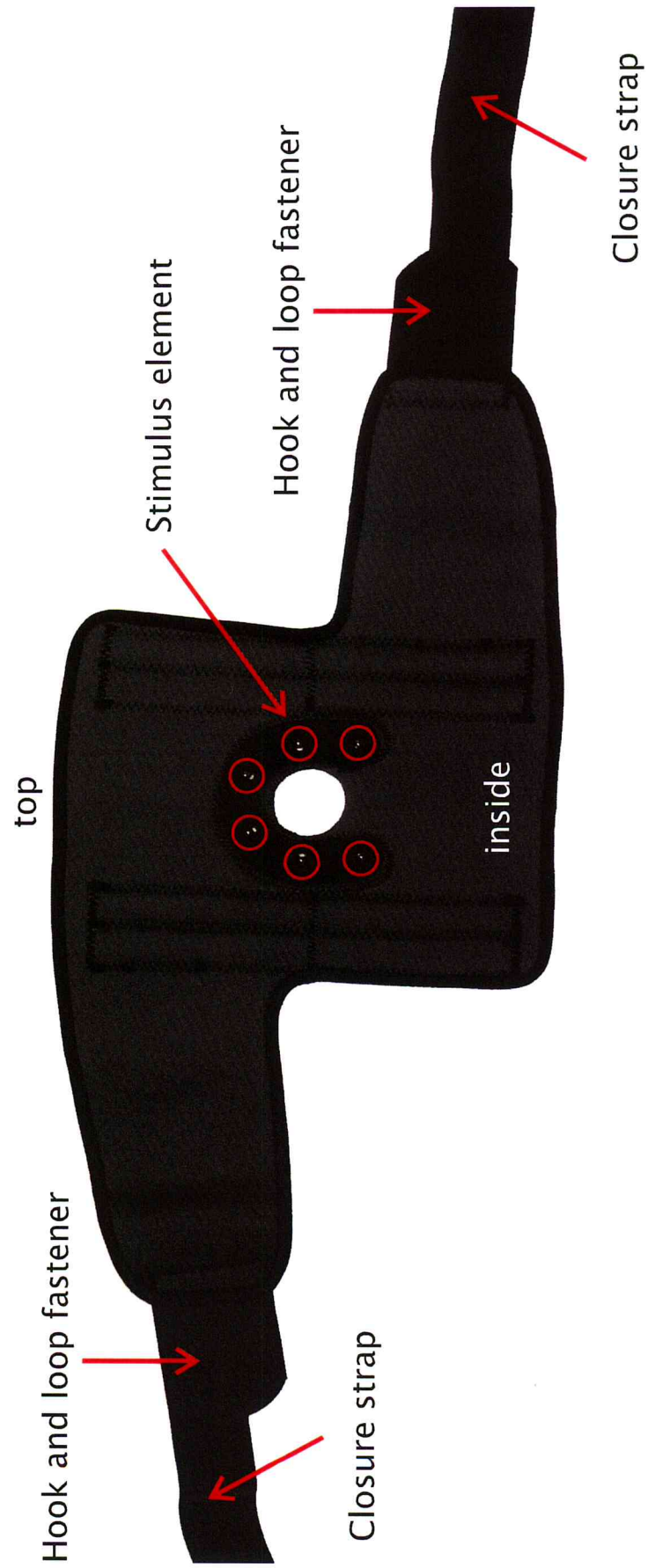

User manual for the knee pain simulator

Positioning and fitting

Make sure that the arrow on the inside of the knee wrap points upwards. Position the knee wrap with the stimulus elements so that the kneecap lies inside the round opening and the stimulus elements are arranged around the kneecap.

Close the knee wrap with the hook and loop fasteners without pulling them too tight. Guide the closure straps around the knee wrap, pull them through the corresponding eyelets and fasten the closure straps to themselves.

If desired, repeat the procedure with the second knee wrap.

Adjusting the intensity

You can adjust the tension of the knee bandages, and thus also the intensity of the simulation, using the closure straps. Tightening the closure straps increases the intensity of the simulation. A loose fitting reduces the intensity.

Cleaning

The knee pain simulator is washable by hand using lukewarm water and mild detergent. Allow to air dry.

Contact

Global Technologies

10388 W. State Road 84, Suite 111,
Davie, Florida USA 33324-4252

Toll-Free: 1-888-437-3900

Intern.: +001-954-746-4270

Fax: 1-954-252-2181

sales@GTsimulators.com

www.GTsimulators.com

General safety information

The simulation of impairments that are aged-related, illness-related or disability-related inevitably involves risks, for instance a higher accident risk.

Impairments should therefore always be simulated with particular caution.

Our products must not be used

- on people with pre-existing impairments due, for instance, to age, illness or disability
- on high-risk individuals, such as pregnant women, epileptics, people with neurological or mental illnesses, people at risk of having a stroke and people with a heart condition
- on people with acute illnesses, such as infections and skin disorders

Increased caution is required when dealing with children and adolescents, as well as people with cognitive impairments or behavioural problems.

Our products may only be used in a safe environment under the supervision of a competent person.

The supervisor is responsible for the product's use. He must ensure that all the aforementioned points are being adhered.

In addition to this general safety information, it is important to comply with the safety information in the relevant user manuals.

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